



## SAFE PATIENT HANDLING – REPOSITIONING DEVICE OPTIONS

Product	Purpose	Other Info
	<b>BED WEDGES (DOUBLE)</b> <ul style="list-style-type: none"> <li>▪ Help to prevent person from falling out of bed.</li> <li>▪ Offers re-positioning options.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Use in conjunction with other bed mobility devices</li> </ul>
	<b>BED WEDGE (SINGLE)</b> <ul style="list-style-type: none"> <li>▪ Provides back support.</li> <li>▪ Can be used for position changes on one side or the other.</li> <li>▪ Maintain trunk elevation for reflux or Gerd.</li> </ul>	<ul style="list-style-type: none"> <li>▪ Use in conjunction with other bed mobility devices</li> </ul>
	<b>GREY SPLIT SHEET</b> <ul style="list-style-type: none"> <li>▪ Turning / Repositioning</li> <li>▪ Attaches to ceiling/floor total lift</li> </ul>	<ul style="list-style-type: none"> <li>▪ Requires 1-2 staff &amp; mechanical total lift (ceiling/floor)</li> <li>▪ remains on the bed</li> </ul>
	<b>FULL POSITONING SLING</b> <ul style="list-style-type: none"> <li>▪ Turning / Repositioning</li> <li>▪ Attaches to ceiling/floor total lift</li> <li>▪ lateral transfers</li> </ul>	<ul style="list-style-type: none"> <li>▪ Requires 1-2 staff &amp; mechanical total lift (ceiling/floor)</li> <li>▪ remains on the bed</li> </ul>
	<b>HOUR GLASS / TRI TURN TURNING DEVICE</b> <ul style="list-style-type: none"> <li>▪ Assists with rolling in bed</li> <li>▪ Can be used as a limb holder</li> </ul>	<ul style="list-style-type: none"> <li>▪ Requires 1-2 staff &amp; mechanical total lift (ceiling/floor)</li> </ul>

## SAFE PATIENT HANDLING – REPOSITIONING DEVICE OPTIONS

	<p><b>NON-FRICTION DEVICE</b></p> <ul style="list-style-type: none"> <li>▪ Turning / Repositioning</li> <li>▪ Moving up/down in bed</li> <li>▪ Flat sheets (2) can assist with donning slings while in bed</li> <li>▪ Moving people out of tight spaces after a fall</li> <li>▪ lateral transfers</li> </ul>	<ul style="list-style-type: none"> <li>▪ Many brands, sizes &amp; colors</li> <li>▪ Available in tubes or flat sheets (with or without handles)</li> <li>▪ Use with a draw sheet</li> <li>▪ Best practice: 2 or more staff</li> <li>▪ Always remove from bed</li> </ul>
	<p><b>EASY GLIDES</b></p> <ul style="list-style-type: none"> <li>▪ Turning / Repositioning</li> <li>▪ Moving up/down in bed</li> <li>▪ Assist with donning slings on beds or in chairs</li> </ul>	<ul style="list-style-type: none"> <li>▪ Multiple sizes and shapes</li> <li>▪ Use with draw sheet</li> <li>▪ 1-2 staff depending on reason or use</li> </ul>
	<p><b>ONE WAY GLIDES</b></p> <ul style="list-style-type: none"> <li>▪ Repositioning in one direction on chairs and beds</li> <li>▪ “locking” mechanism to reduce sliding forward in chairs/down in bed</li> </ul>	<ul style="list-style-type: none"> <li>▪ Multiple sizes and shapes</li> <li>▪ 1-2 staff depending on reason or use</li> <li>▪ Can sew smaller ones together for bed repositioning</li> </ul>
	<p><b>FINGER LADDER</b></p> <ul style="list-style-type: none"> <li>▪ Helps person to assist with turning, positioning, sitting up</li> </ul>	<ul style="list-style-type: none"> <li>▪ Many brands &amp; styles</li> </ul>
	<p><b>SIDE RAILS</b></p> <ul style="list-style-type: none"> <li>▪ Helps person to assist with turning, positioning, sitting up, laying down</li> </ul>	<ul style="list-style-type: none"> <li>▪ Different styles, sizes and mounting options</li> <li>▪ Some facilities have specific side rail requirements for safety, check your facility requirements first</li> </ul>
	<p><b>TRENDELENBERG POSITION</b></p> <ul style="list-style-type: none"> <li>▪ Use in conjunction with soft goods to move person up in bed</li> </ul>	